Swiss Premium Beds

_75RF bridge®
_75RF

0408130344

Bedienungsanleitung
Mode d'emploi
Operating instructions
Gebruiksaanwijzing
Modo de empleo
Istruzioni d'uso
使用说明书

DE Herzlichen Glückwunsch!

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Motor control

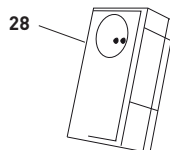
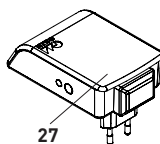
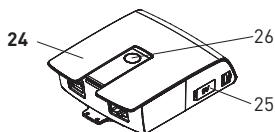
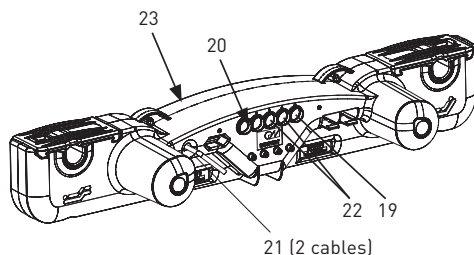
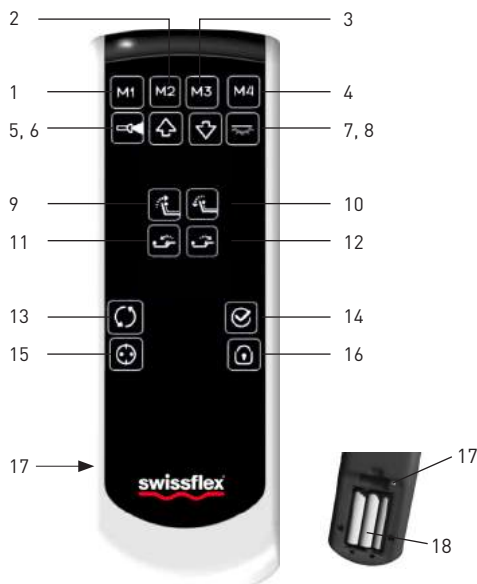
- 19 Sync cable connection
- 20 Emergency light (internal)
- 21 Connection for mains supply unit / Con-
nection for external under-floor lighting
- 22 Connection for auxiliary motors
- 23 Emergency light (internal, back of the
motor)

24 Switching power supply

- 25 Battery compartment for emergency lowe-
ring (2 x 9V)
- 26 Emergency lowering button / teach-in
button

27 Bluetooth RF-Gateway (optionally)

28 User socket (optionally)

Health and comfort

Welcome to the world of Swissflex!

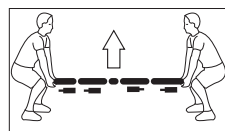
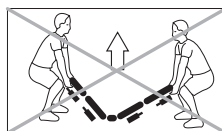
Your Swissflex _75RF / _75RF bridge® is a Swiss quality product. If you follow the simple operating and safety instructions, it will function to your complete satisfaction for many years. In addition to its many other advantages, your _75RF / _75RF bridge® offers an undisturbed sleep environment. All electrical connections between the mains plug and the bed are virtually free of mains voltage (below 0.5 watts) thanks to EcoTec.

Safety instructions

This device can be used by children aged 8 and above and by people with reduced physical, sensory or mental capabilities or people lacking experience and knowledge, provided that they are supervised or have been instructed in the safe use of the device and understand the dangers that may arise. Children must not play with the device. Cleaning and user maintenance must not be carried out by children without supervision.

CAUTION! The uni 14_/15_/22_/20 _75RF / _75RF bridge®RF / _75RF / _75RF bridge®RF bridge® and accessories must be protected from rain and other moisture to prevent causing a fire hazard or the risk of electric shock. In case of spillages on an accessory, the mains cable must be disconnected from the power outlet and checked by your Swissflex stockist.

When installing the slat base, ensure that the head, back and

foot sections are not obstructed by furniture.

Ensure that the mains supply is disconnected before any assembly or disassembly work is carried out on the slat base or the bed frame.

Do not connect any devices to the user socket **(28)** that could endanger persons or damage objects when the socket is turned on by radio remote control, in particular electrical tools such as saws, drills and similar (risk of injury) or portable heating devices (risk of fire).

Never place the mains cable between the user socket **(28)** and the bed in such a way that the cable insulation could be damaged by sharp edges or twists. Never feed the cable through the adjustment mechanism.

In the case of damage to cables, to the switching power supply housing or to the user socket **(28)**, do not adjust the bed to a seated position. Remove the mains plug from the socket and inform your stockist.

Never open the user socket housing **(28)** to which the cables

are connected. If you interfere with any electrical part of the user socket **(28)**, the guarantee entitlements will no longer apply.

Handle the uni 14_/15_/22_/20 _75RF / _75RF bridge®RF / _75RF / _75RF bridge®RF bridge® and accessories with care; do not disassemble or modify them, do not allow children to play with them and prevent frail individuals from using the product without supervision.

The product may not be used with a medical device or be installed in a medical device.

The product may not be used in an environment in which flammable or explosive gases or vapours (e.g. anaesthetics) are likely to occur.

The product may not be used to raise and lower loads.

Only use spare parts that have been produced or approved by Swissflex. Only these parts guarantee adequate safety.

The slat base may only be used with the power supply provided.

Initial setup «Touch»

Place the batteries into the radio remote control

Remove the cover from the battery compartment **(17)** on the back of the radio remote control. Place 3 AAA-batteries **(18)** into the compartment, paying attention to the +/- markings. Close the compartment cover again. The maximum range of the radio remote control is 15 metres.

Teaching-in the radio remote control (already taught-in in the factory)

1. Remove the radio remote control's battery cover on the back of the device.
2. Insert / check the batteries (3 x AAA).
3. Activate the teach-in button **(26)** on the switching power supply (press 2 x).
4. A small red button is located above the batteries in the battery compartment of the device **(17)**.
5. Press the button; a blue LED will start flashing.
6. Hold the button down until the blue LED is permanently illuminated; this means that pairing has been completed.

Bluetooth®

1. Download the standard „Swissflex“ app from the relevant platform for Android (Google play) or Apple iOS (App Store).
2. Install the standard „Swissflex“ app onto your Smartphone / Tablet PC.
3. Installation / Initial setup, see manual of the Bluetooth RF-Gateway.

Synchro function (with sync cable)

When two _75RF / _75RF bridge® are linked together with a sync cable (connection jack **(19)**), the synchro function **(13)** is required. By pressing and holding down the button **(13)**, the synchro function is activated/deactivated. When pressing the travel button **(6-7)**, the control systems synchronise and then move in the confirmed direction of travel. After successful activation, the button glows blue continuously. After successful deactivation, the button appears white. After plugging in the sync cable, carry out a reset as follows:

- Press down the 'Memory/Save' button **(14)** for 3 seconds.
- The M1-M4 buttons will first light up blue and then switch to white again.
- The LCD backlighting system will flash white once after several seconds.
- Immediately press down the synchro button **(13)** and keep it pressed down. The control systems power down.
- If the entire backlighting system flashes white twice, you can release the synchro button **(13)**.

The internal meter readings for the individual motors have now been reset to '0' and the sync function can work correctly.

Under-floor lighting

Activate the under-floor lighting by pushing the „Under-floor lighting“ button **(8)**. This switches off automatically after 30 minutes, or if the button is pushed once again. This button is illuminated (blue) when it is pushed.

Pocket lamp

The pocket lamp is illuminated when the „Pocket lamp“ button **(5)** is held down. The light turns off when the button is released. This button is illuminated (blue) while the pocket lamp is in use.

Lock buttons

To lock the buttons, hold down the „Lock buttons“ button **(16)** for five seconds. Once the button lock has been activated, all buttons (except for the mains outlet **(15)**, the pocket lamp **(5)**, the under-floor lighting **(8)** and the button lock **(16)**) flash (blue) twice.

All buttons flash twice (white) when a disabled button is pushed. The lock is disabled by holding down the „Lock buttons“ button **(16)** for five seconds. All buttons flash twice (blue) if this was successful.

Adjustment options «Touch»

The sleep and sit positions have been pre-programmed in the factory.

- M1 Read and watch TV without getting tired
- M2 Relieving position for colds, respiratory disorders or cardiovascular complaints
- M3 Ergonomic position for backaches and vertebral disk complaints
- M4 Aids for getting into and out of the bed

The positions can be overwritten individually.

Resetting the factory-programmed memory positions

Hold down the memory button **(14)** for 3 seconds – the background lighting then flashes once, the under-floor lighting button **(8)** must then be pressed within a second. The entire RF-Touch lighting flashes twice to confirm that the factory settings have been reset.

Saving the sitting and lying position to a memory button

1. Press the memory button **(14)** 3 x within 2 seconds.
2. The memory button is illuminated (blue) **(1–4)**.
3. Press the desired memory button for memory position 1–4 **(1–4)** 1x within three seconds.

The programming of a new position automatically overwrites an old position. A successful save is visualised by a change in colour of the selected memory button LED (white). All memory buttons turn white two seconds after the memory button is released.

To call up a position stored in memory

Press the desired memory button for memory position **(1–4)**, until all the comfort sections have reached the programmed position. The activated memory button is illuminated (blue) during the travel time.

To reset to the horizontal position

Press the “all motors down” button **(7)** to simultaneously move all sections to the horizontal position.

Care and Maintenance «Touch»

Your _75RF / _75RF bridge® is a high-quality product that has been designed to serve you reliably for many years to come. For occasional cleaning and the removal of dust, we recommend the use of a dry cloth.

Changing the batteries in the radio remote control

Remove the cover from the battery compartment **(17)** on the back of the radio remote control. Place three AAA batteries into the compartment, paying attention to the +/- markings. Close the compartment cover again.

Changing the batteries (25) in the _75RF / _75RF bridge® switching power supply (batteries for emergency lowering)

Carefully remove the batteries from the battery clips. Place two 9V block batteries in the compartment, paying attention to the +/- markings. The batteries in the motor provide power in the case of a power failure.

Troubleshooting «Touch»

In case of malfunctions with your _75RF / _75RF bridge®, consult the following troubleshooting tips. If the problem cannot be resolved, disconnect the _75RF / _75RF bridge® from the mains outlet and contact your Swissflex stockist.

Loss of power in the sitting position; emergency lowering button

The switching power supply contains two bloc batteries **(25)** to revert the bed to the horizontal (sleeping) position in the event of a power failure while sitting up in the bed:

Hold the emergency lowering button **(26)** down until all sleeping surfaces are in the horizontal position. Once voltage has been restored, wait at least 10 seconds before resetting the system by completing the following steps:

- Press down the 'Memory/Save' button **(14)** for 3 seconds.
- The M1-M4 buttons will first light up blue and then switch to white again.
- The LCD backlighting system will flash white once after several seconds.
- Immediately press down the synchro button **(13)** and keep it pressed down. The control systems power down.
- If the entire backlighting system flashes white twice, you can release the Synchro button **(13)**.

The internal meter readings for the individual motors have now been reset to '0' and the stored memory positions are correct again.

Use of the emergency lowering button consumes a great deal of battery power, which means you will need to replace the batteries.

The _75RF / _75RF bridge® does not work despite being switched on and all components being connected properly

Disconnect the mains supply unit from the mains outlet. Wait a minute and then reconnect it. If necessary, connect a bedside lamp to check that mains voltage (240 V) is present. If there is no current, check the fuse.

Press the "all motors down" button **(7)** to reset all adjustments. The control of the _75RF / _75RF bridge® contains microprocessors, so irregular power supply can lead to malfunctions.

The radio remote control does not work (The back-ground light does not work)

Insert new batteries. Where applicable, remove any obstacles between the radio remote control and the motor control.

After a power failure once power has been restored to the system

To restart the initial setup of the _75RF / _75RF bridge®, press button **(26)** on the switching power supply **(24)** until all of the motors have powered down and this is confirmed by the motor's emergency light blinking twice (can take 20 seconds).

Technical data

- Supply voltage:** 110-240 V
- Remote control power supply:** 3 x AAA Batteries
- Power rating motors:**
 - Intermittent operation from 2 min / 18 min
- Power rating user socket :** 2000 W / 10 A
- Thrust motors:** 2 x 6000 N
- Power supply for emergency lowering:**
 - 2 x 9 V block battery

Problem	Possible cause	Solution
Remote control or control system not working.	No mains supply.	Restore mains connection.
	Manual switch or control system defective.	Contact your supplier / stockist.
Control systems suddenly no longer travel / move.	Thermoswitch on the transformer may have been triggered or may be defective.	Leave the control system in the rest position for 20-30 minutes.
	Thermal cut-out in the transformer may have been triggered or may be defective.	Contact your supplier / stockist.
	Internal fuse may have been triggered or may be defective.	Contact your supplier / stockist.
	No mains supply.	Restore mains connection.
	Supply line (mains and / or manual switch / auxiliary control system) interrupted.	Check the supply line and restore contact if applicable.
The battery-operated electric reset function cannot be activated.	Block battery / block batteries flat.	Check and replace the block battery / block batteries if necessary.
	Block battery/batteries not connected.	Connect the block battery / block batteries.

Radio remote control «Classic»

- 1 Control system back section «up»
 - 2 Control system back section «down»
 - 3 Control system thigh section «up»
 - 4 Control system thigh section «down»
 - 5 All motors «up»
 - 6 All motors «down»
 - 7 Pocket lamp
 - 8 Battery compartment (on the back of the radio remote control)
 - 9 3 x AAA-Battery (included in the delivery)
- 1 + 2 Floor lighting
3 + 4 User socket on / off
6 + 7 Activating / disabling the sync function

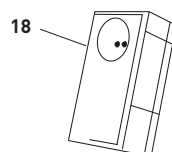
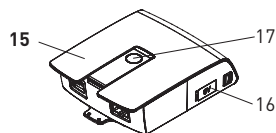
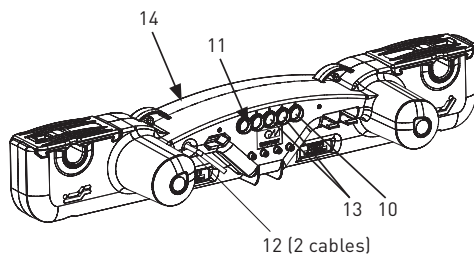
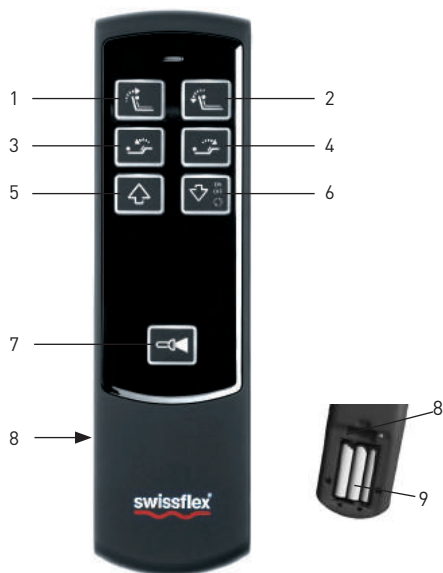
Motor control

- 10 Sync cable connection
- 11 Emergency light (internal)
- 12 Connection for mains supply unit / Connection for external under-floor lighting
- 13 Connection for auxiliary motors
- 14 Emergency light (internal, back of the motor)

15 Switching power supply

- 16 Battery compartment for emergency lowering (2 x 9V)
- 17 Emergency lowering button / teach-in button

18 User socket (optionally)

Initial setup «Classic»

Place the batteries into the radio remote control

Remove the cover from the battery compartment **(8)** on the back of the radio remote control. Place 3 AAA-batteries **(9)** into the compartment, paying attention to the +/- markings. Close the compartment cover again. The maximum range of the radio remote control is 15 metres.

Teaching-in the radio remote control (already taught-in in the factory)

1. Remove the radio remote control's battery cover on the back of the device.
2. Insert / check the batteries (3 x AAA).
3. Activate the teach-in button **(17)** on the switching power supply (press 2 x). Now the motor's emergency light turns on.
4. Press buttons **(1+2)** simultaneously on the radio remote control until the motor's emergency light turns off. This means that pairing has been completed.

Synchro function (with sync cable)

When two _75RF / _75RF bridge® are linked together with a sync cable (connection jack **(10)**), the synchro function is required. By pressing and holding down the button **(6+7)**, the synchro function is activated/deactivated. When pressing the travel button **(1-4)**, the control systems synchronise and then move in the confirmed direction of travel. After plugging in the sync cable, carry out a reset as follows:

Press button **(17)** on the switching power supply **(15)** until all of the motors have powered down and this is confirmed by the motor's emergency light blinking twice (can take 20 seconds).

The internal meter readings for the individual motors have now been reset to '0' and the sync function can work correctly.

Under-floor lighting

Activate the under-floor lighting by pushing the „Under-floor lighting“ buttons **(1+2)**. This switches off automatically after 30 minutes, or if the button is pushed once again.

Pocket lamp

The pocket lamp is illuminated when the „Pocket lamp“ button **(7)** is held down. The light turns off when the button is released.

Adjustment options «Classic»

You can individually adjust the back section or the thigh section by using the buttons **(1-4)**.

All motors can be simultaneously put in position «up» or «down» with the buttons **(5)** and **(6)**.

To reset to the horizontal position

Press the “all motors down” button **(6)** to simultaneously move all sections to the horizontal position.

Care and Maintenance

«Classic»

Your _75RF / _75RF bridge® is a high-quality product that has been designed to serve you reliably for many years to come. For occasional cleaning and the removal of dust, we recommend the use of a dry cloth.

Changing the batteries in the radio remote control

Remove the cover from the battery compartment **(8)** on the back of the radio remote control. Place three AAA batteries into the compartment, paying attention to the +/- markings. Close the compartment cover again.

Changing the batteries in the uni 14/_15/_22/_20 _75RF / _75RF bridge®RF / _75RF / _75RF bridge®RF bridge® switching power supply **(15)** (batteries for emergency lowering)

Carefully remove the batteries from the battery clips. Place two 9V block batteries in the compartment, paying attention to the +/- markings. The batteries in the switching power supply provide power in the case of a power failure.

Troubleshooting «Classic»

In case of malfunctions with your uni 14/_15/_22/_20 _75RF / _75RF bridge®RF / _75RF / _75RF bridge®RF bridge®, consult the following troubleshooting tips. If the problem cannot be resolved, disconnect the _75RF / _75RF bridge® from the mains outlet and contact your Swissflex stockist.

Loss of power in the sitting position; emergency lowering button

The switching power supply **(15)** contains two bloc batteries to revert the bed to the horizontal (sleeping) position in the event of a power failure while sitting up in the bed:

Hold the emergency lowering button **(17)** down until all sleeping surfaces are in the horizontal position. Once voltage has been restored, wait at least 10 seconds be-

fore resetting the system by completing the following steps:

Press button **(17)** on the switching power supply **(15)** until all of the motors have powered down and this is confirmed by the motor's emergency light blinking twice [can take 20 seconds].

The internal meter readings for the individual motors have now been reset to '0' and the sync function can work correctly again.

Use of the emergency lowering button consumes a great deal of battery power, which means you will need to replace the batteries.

The _75RF / _75RF bridge® does not work despite being switched on and all components being connected properly

Disconnect the mains supply unit from the mains outlet. Wait a minute and then reconnect it. If necessary, connect a bedside lamp to check that mains voltage (240V) is present. If there is no current, check the fuse. Press the "all motors down" button **(6)** to reset all adjustments. The control of the _75RF / _75RF bridge® contains microprocessors, so irregular power supply can lead to malfunctions.

The radio remote control does not work (The back-ground light does not work)

Insert new batteries. Where applicable, remove any obstacles between the radio remote control and the motor control.

After a power failure once power has been restored to the system

To restart the initial setup of the uni 14_/15_/22_/20 _75RF / _75RF bridge®RF / _75RF / _75RF bridge®RF bridge®, press button **(17)** on the switching power supply **(15)** until all of the motors have powered down and this is confirmed by the motor's emergency light blinking twice. [Can take 20 seconds].

Technical data

- Supply voltage:** 110-240 V
- Remote control power supply:** 3 x AAA Batteries
- Power rating motors:**
Intermittent operation from 2 min / 18 min
- Power rating user socket :** 2000 W / 10 A
- Thrust motors:** 2 x 6000 N
- Power supply for emergency lowering:**
2 x 9 V block battery

Problem	Possible cause	Solution
Remote control or control system not working.	No mains supply.	Restore mains connection.
	Manual switch or control system defective.	Contact your supplier / stockist.
Control systems suddenly no longer travel / move.	Thermoswitch on the transformer may have been triggered or may be defective.	Leave the control system in the rest position for 20-30 minutes.
	Thermal cut-out in the transformer may have been triggered or may be defective.	Contact your supplier / stockist.
	Internal fuse may have been triggered or may be defective.	Contact your supplier / stockist.
	No mains supply.	Restore mains connection.
	Supply line (mains and / or manual switch / auxiliary control system) interrupted.	Check the supply line and restore contact if applicable.
The battery-operated electric reset function cannot be activated.	Block battery / block batteries flat.	Check and replace the block battery / block batteries if necessary.
	Block battery/batteries not connected.	Connect the block battery / block batteries.

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Recticel Bedding (Schweiz) AG